

hummate

hummate Body Massager

Function

- Rolling
- Vibration
- Kneading



HPC-11001



Designed & Developed by Happyroom co.,Ltd. Korea

**TO ALLOW YOUR BACK, NECK & SHOULDER AND HIP
TO RELAX WITH COMFORTABLE
STRONG KNEADING MASSAGE**

www.huemate.co.kr

hummate

Function


- Rolling
- Vibration
- Kneading

SPECIFICATION

Rated Voltage: DC 12V 4A
 Power Consumption: 48W
 Cover Material: Polyurethane
 Major Function: Shiatsu with heat
 Operating Time: 15 Minutes
 Net Wt / Gross Wt: 11.8 kg / 13.3 kg
 Dimensions: 78.5 X 19 X 45.5 (cm)
 Manufacturer: Happyroom Co., Ltd.
 Electrical Safety Authorization (Adapter): HU10539-12010
 Origin : Made in China

HOW TO USE

1. To place the Body Massager on to a Sofa or a Chair
2. To plug in to an electrical outlet after connecting cable to adapter cord
3. To push Start button
4. To operate Neck massage
5. To operate Heat function
6. To operate Vibration function
7. To select 3 operation methods
8. To select 3 manual operation methods
9. To operate Back massage
10. The operation time is 15 minutes. It stops automatically after 15 minutes in use.
11. To plug out after use

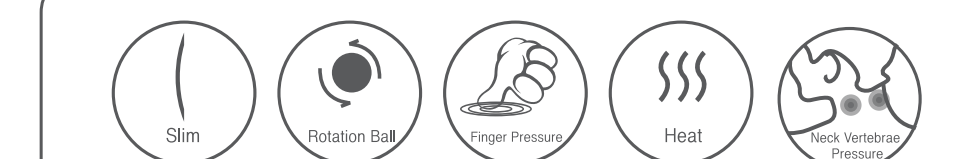

- The thickness of main body is only 2 cm with super slim design.
- The height of neck kneading massage part can be adjusted in 3 step (by manual).
- Back massage has 3 sections (upper, middle and lower back).
- To select 3 operation modes (V, H, C).
- It stops automatically in 15 minutes after operation.
- It requires DC 12V electrical power.

Manufacture Date: mark separately
 Manufacturer: Happyroom Co., Ltd. Korea
 Designed by Happyroom Co., Ltd. Korea
 1 year limited warranty
 Warranty is not applicable for the damage by abuse



PRECAUTION

This device is not recommended to use for those who have such serious diseases as osteoporosis, malignant tumor and disk patient etc. For other patients, it is required to use the device after consultation with doctor.

